




# HemaLimit

Absorbable Haemostat  
Gelatin Sponge Powder

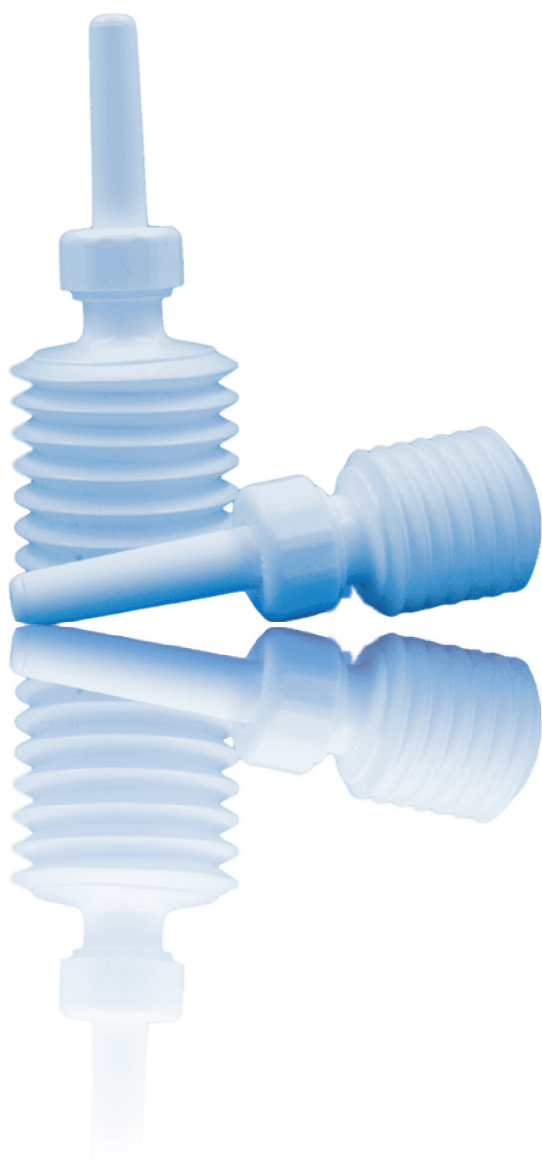

## Special Characteristics

◆ **HEMALIMIT Powder** is indicated for surgical procedures (except ophthalmic) for haemostasis, when control of capillary, venous and arteriolar bleeding by pressure, ligature and other conventional procedures is ineffective or impractical.

◆ **HEMALIMIT Powder** is a sterile, absorbable gelatin powder intended for haemostatic use by applying to a bleeding surface. The powder is water- insoluble and off-white in appearance. The modified Gelatine based, absorbable, natural polymer particular powder accelerate the clotting process to achieve haemostasis quickly and effectively in 2 minutes.

◆ **HEMALIMIT Powder** made from 100% pharmaceutical Gelatin. It is biodegradable and biocompatible with inherent proagulant activity and it is entirely eliminated from the body in < 4 weeks.



# HemaLimit

Absorbable Haemostat  
Gelatin Sponge Powder

## Product Features

### Effective

- The haemostatic effect of Hemalimit Powder is almost immediate, the full haemostasis is achieved by 2 minutes after application.
- Does not leave voids between clot material and tissue surfaces
- Better blood absorption capacity than MPH powder with significantly lower chances of rebleeding
- Indicated for general use for capillary, venous and arteriolar bleeding
- Better mechanical integrity to the clot providing more fibrin matrix within clots
- Easy to apply to large areas and irregular surfaces

### Safe

- Made from 100% pharmaceutical Gelatin
- Entirely eliminated in < 4 weeks, there are no deposits left behind the tissues and cells
- No risk of encapsulation and overdose
- PH-neutral can be combined with drugs as indicated
- Can be used in patients with anticoagulant therapy
- Natural, safe, biodegradable and biocompatible with inherent procoagulant activity

### Easy to Use

- Easy to apply to large areas and irregular surfaces
- NO mixing
- NO assembly
- Room Temperature Self Storage
- 3 Years Shelf Life





# HemaLimit

Absorbable Haemostat  
Gelatin Sponge Powder

## Indication/Clinical Experience

- ◆ General Surgery
- ◆ Cardiac Surgery
- ◆ Orthopaedic and Trauma Surgery
- ◆ Spinal Surgery
- ◆ Plastic Surgery
- ◆ Gynaecology and Caesarean Section
- ◆ ENT Surgery

- ◆ Visceral and Transplantation Surgery
- ◆ Vascular and Cardiothoracic Surgery
- ◆ Neurosurgery
- ◆ Tumour Surgery
- ◆ Breast Surgery
- ◆ Oral and Maxillofacial Surgery
- ◆ Minimal and Robot assisted Surgery

## Easy Application



Remove all excess blood from the intended site by blotting, wiping, or suctioning. Removing excess blood allows HEMALIMIT Powder to make direct contact with the site



Immediately apply a generous amount of HEMALIMIT powder



For profuse bleeding, apply moderate compression to the source of bleeding using a dry compress. Irrigation with saline before carefully removing the gauze is recommended.



Moisten the compress with sterile saline solution before removal and remove it carefully. If bleeding continues, remove excess particles, and repeat the procedure.



# HemaLimit

Absorbable Haemostat  
Gelatin Sponge Powder

## Specifications

Product Ref. No	Product Name	Weight/Volume	Number of Pieces per box
HL-02B	HEMALIMIT	2 Grams	1 unit/Box
HL-03B	HEMALIMIT	3 Grams	1 unit/Box



Produced exclusively for

**EOS**  
MEDICAL SOLUTIONS

10, Kassiopis Str, 17237  
Ymittos, Attika  
Email: [info@eosmed.eu](mailto:info@eosmed.eu)  
Website: [www.eosmed.eu](http://www.eosmed.eu)  
GREECE

**CE**  
2460